



德康醫療

MedOne Medical



Trauma Leg

Task Trainer





Trauma Leg Task Trainer


Practice first aid measures and manage a major hemorrhage with the Trauma Leg.

The trainer represents an injured leg with laceration, bullet entry, and bullet exit wounds.

The bleeding system enables the flow of blood from the wounds.

You can apply direct pressure, pack the wound with gauze, place a pressure dressing, and apply the tourniquet to really stop the bleeding.

Made with a soft silicone coating, the Trauma Leg reproduces the feeling of touching the human skin.

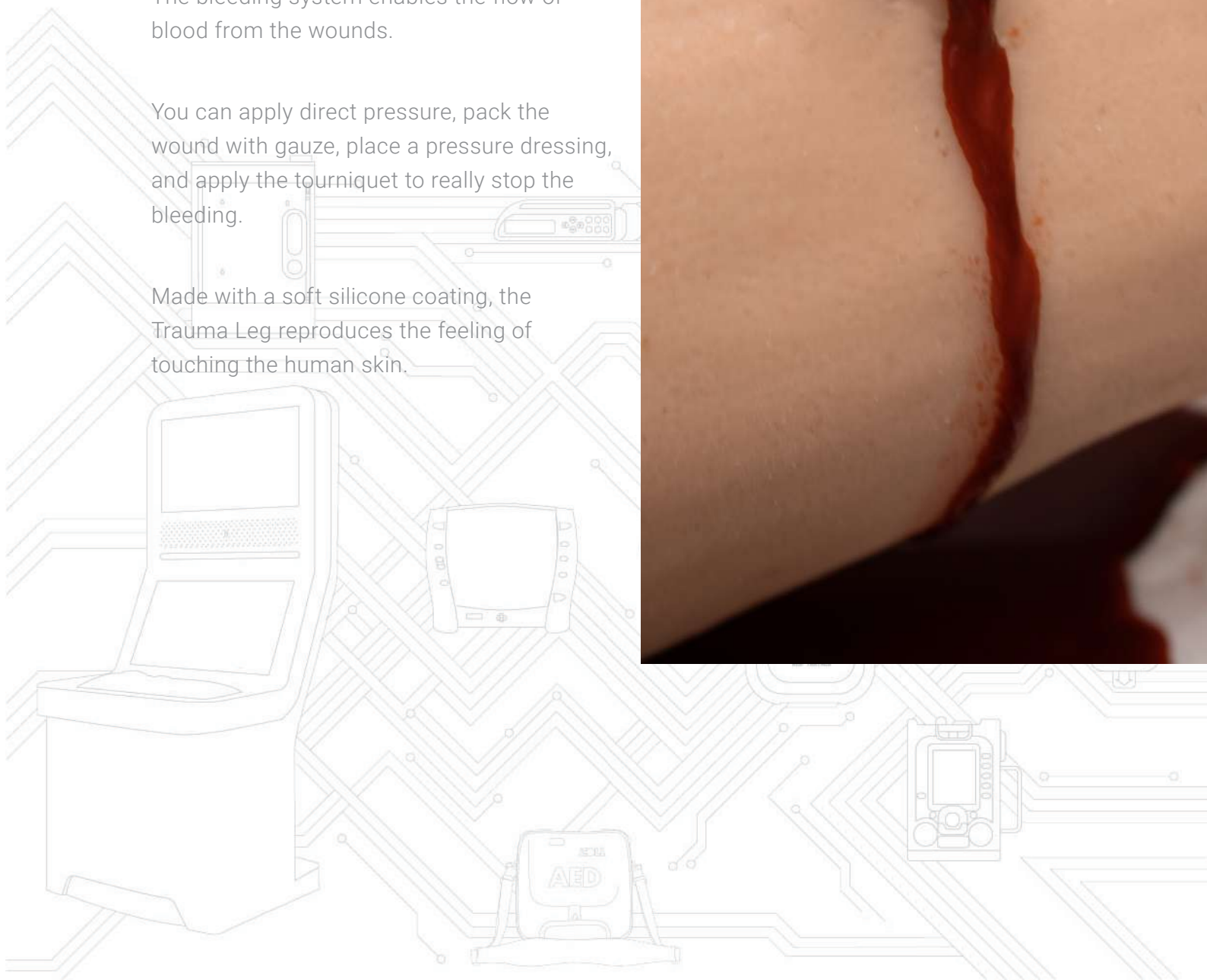



What you need



Simulated blood
INCLUDED



Application bottle
INCLUDED



Tourniquet
NOT INCLUDED



Preparation



Step 1: Insert the bottle nozzle into one of the tubes at the top side of the Trauma Leg. Each tube is connected with one of the three wounds.

Step 2: Fill the bottle with simulated blood.

Step 3: Attach the application bottle to the

Trauma Leg by gently inserting the bottle's nozzle into one of the tubes at the top side of the trainer. Each tube is connected with one of the three wounds.

Step 4: Apply some force to make sure the nozzle is secured tightly.

Step 5: Squeeze the application bottle to push the simulated blood through the tubes into the wound.



Detachable tubes

